


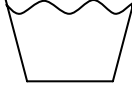


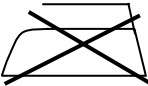
COLORS - Laundering Recommendations

Patient Apparel, Scrubs, Uniforms, Lab Coats, Bibs, Mattress Pads, Incontinents, Spreads, Blankets

 WASHING	Pre-Rinse (x1)	Water temperature: 160°F Alkalinity ppm: 200 – 400ppm Alkalinity pH: 11.0 – 12.0pH Final Rinse pH: 5.5 – 6.5pH
	Detergent (x1)	
	Post-Rinse (x2-3)	
	Sour / Softener	

 NO BLEACH	DO NOT use bleach
---	--------------------------

 LIMITED DRYING	Apparel & Bibs: Tumble Dry - MAX 180°F Mattress Pads, Spreads: Tumble Dry – MAX 160°F Blankets & Incontinents
--	---

 NO IRONING	DO NOT flat-work iron
--	------------------------------

All Products

Must be laundered prior to use
Only wash with like items
Only wash with like colors

Scrubs & Gowns

Must pre-rinse up to 3 times at 160°F when new

Mattress Pads & Blankets, Spreads

May require extra post-rinses and
Sensitive to Hot temperatures

Velcro Bibs

Must fasten Velcro together before laundering

Chemicals

Quantity will vary with machine size and type
Confirm with your chemical vendor for correct quantity