
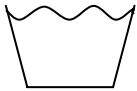


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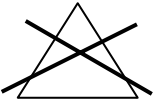



# SURGICAL- Laundering Recommendations

## Reusable Gowns: Gore, Blockade, Stericloth, Resistat

 <b>WASHING</b>	Pre-Rinse (x2)	Water temperature: 160°F
	Detergent (x1)	Alkalinity ppm: 200 – 400ppm
	Carry (x1)	Alkalinity pH: 10.5 – 11.0pH
	Post-Rinse (x4)	Nonionic Surfactant Cloud at 160°F
	Sour (x1)	Final Rinse pH: 5.5 – 6.5pH
		<b>DO NOT</b> use softener

 <b>NO BLEACH</b>	<b>DO NOT</b> use bleach - will damage barrier properties
--	---

 <b>DRYING</b>	Tumble Dry - <b>MAX</b> temperature 170°F Remove or Autoclave promptly
--	---

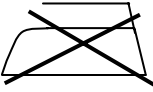 <b>NO IRONING</b>	<b>DO NOT</b> flat-work iron
--	------------------------------

### All Products

Must be laundered prior to use  
Only wash with like items  
Only wash with like colors

### Chemicals

Quantity will vary with machine size and type  
Confirm with your chemical vendor for correct quantity

### Quality Assurance

Must be used to test effective product performance